

Cytology and Surgical Pathology Containers

Liquid-based PAP (GYN)

Use either brush/spatula or broom-like device to collect specimen. Follow directions provided with supplies. In addition to PAP testing, the following tests can be performed from the same vial:

- Chlamydia
- Gonococcus (GC)
- Human Papilloma Virus (HPV)
- Candida
- Gardnerella
- Trichomonas
- Leukorrhea Panel



Specimen Container Code **LQ**

Non-GYN PAP

Cytology preservative for non-GYN specimens such as urine, thyroid aspirates and other fluids. Equal volume of fluid and preservative should be maintained.



Specimen Container Code **NGYN**

Conventional PAP collection kit

Includes slide, cervical brush, spatula, and cotton tip applicator. Specimens should be preserved with spray fixative immediately after collection



Specimen Container Code **CPAP**

Formalin containers

10% buffered formalin in either 15 ml or 60 ml size. Used to preserve tissue for histologic exam (surgical pathology). Hollandes (green) fixative is no longer required for gastrointestinal biopsies.



Specimen Container Code **FRM**



**CONSOLIDATED
LABORATORY SERVICES**

St. Vincent's HealthCare

**For all your laboratory needs
308-5600**

REV: 0114